

# Favosept® NF

## Concentrate



## Disinfectant for medical and rotary instruments

- No alcohols or aldehydes
- Gentle on materials
- Very good cleaning properties
- Ideal for ultrasonic baths
- Biodegradable
- Non-corrosive

### Extensive efficacy spectrum:

- Bactericidal
- Tuberculocidal
- Levurocidal
- Limited virucidal (inactivates enveloped viruses such as HBV, HIV, HCV)

### Range of application:

**Favosept® NF Concentrate** is a high-quality quaternary bond-based disinfectant for instruments with excellent cleaning power, especially for ultrasonic baths. This high-performance product is used for surgical and dental instruments of all kinds.

**Favosept® NF Concentrate** has short exposure times – even against high concentration of germs – and provides comprehensive effectiveness.

### Instruction for use:

Insert used instruments in **Favosept® NF** bath immediately after use, in order that contamination does not dry onto the surface. Strongly contaminated instruments must be pre-cleaned with water before disinfection. The instruments have to be fully inserted in the solution. After disinfection, thoroughly rinse the instruments with water and let the instruments dry. Do not mix the solution with other disinfectants or cleansing agents. Depending on the contamination level, the solution should be regularly replaced.

Can also be used in the ultrasonic bath.

### Active ingredients:

100g of **Favosept® NF Concentrate** contain:  
 3,3 g Bis (3-aminopropyl)dodecylamine  
 2,8 g Didecylmethylpolyoxyethylammonium propionate, non-ion surfactant

### Exposure time:

#### Bacteria and Candida albicans

in acc. to VAH/ DGHM:	3,0%	15 min
- dirty conditions	2,0%	30 min
	1,0%	60 min

#### In ultrasonic bath:

- dirty conditions	3,0%	5 min
--------------------	------	-------

**TbB u. Mycobacterium terrae:** 3,0% 60 min

#### Enveloped viruses (e.g. HBV, HIV, HCV)

in acc. to DVV/ RKI:	3,0%	15 min
	1,0%	60 min

Unit	Article.-No.
1 l metered bottle	163
5 l canister	164

On request we would be pleased to submit tailored conditions.

+49 (0) 721 62686-0 • info@favodent.com

CE 0482 N-28100

Instrument Disinfection